



Euron® Wings Fitting Guide

Euron®
The Same **But Better**

Euron® Wings

Belted Pads

Euron Wings is an anatomically shaped pad with a resealable belt and waistband, designed for moderate to heavy incontinence.

Available in a range of sizes and absorbencies, Euron Wings features our all-white CottonFeel textile backsheet.

Fitting Instructions

Step one

- 1 Unfold the product and open out the belt, first the right side and then the left.

Halve and fold the pad lengthways, cupping the product in the middle.



Step two

- 2 Starting from the front, insert the pad between the legs, then arrange and smooth the product over the abdomen and buttocks.



Step three

- 3 Place the product against the client's back, at waist height. Wrap the belt around the waist, securing it at the front to create a snug fit.



Step four

- 4 Bring the product upwards from between the legs and apply the belt to the waistband. Ensure that the product is fitted securely and comfortably between the legs at the groin area.



Important Advice

The use of talcum powder is not recommended, oil based barrier creams may reduce the performance of Euron Wings pads. If barrier creams must be used they should be applied sparingly and only on treatable areas.

Used products should be disposed of in the appropriate manner. They cannot be flushed down the toilet.



Euron Wings Sizes

| | |
|--------|------------|
| Medium | 70 - 115cm |
| Large | 95 - 135cm |

Contact Us

ONTEX HEALTHCARE UK LTD
2nd Floor, Original Wing, Weldon House
Corbygate Business Park, Priors Haw Road
Weldon North, Corby, Northamptonshire
NN17 5JG

TEL: 01536 269744

FAX: 01536 400134

WEB: www.euronuk.com

Euron[®]
The Same **But Better**

Ontex