



# **Euron**® Form Fitting Guide

**Euron**®  
The Same **But Better**

# Euron® Form

## All-In-Ones

Euron Form all-in-ones are designed to meet the needs of individuals with moderate to heavy incontinence.

Euron Form is available in a wide range of sizes and absorbency levels to ensure client comfort. The product also features resealable tabs to enable re-application if required.

## Fitting Instructions

### Step one

- 1 Hold each end of the Euron Form all-in-one and gently pull it open to activate the anti-leak cuffs.


Halve and fold the all-in-one lengthways, cupping the product in the middle.



### Step two

- 2 If the client is immobile begin by gently rolling them onto their side.

Starting from the front, insert the all-in-one between the client's legs, then arrange and smooth the pad over the abdomen and buttocks.

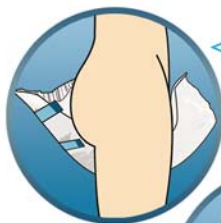


Immobile >




## Step three

- 3** Secure both bottom tabs in position, ensuring a secure fit around the leg. The all-in-one requires a close fit at the groin. The tabs can be fixed in either an angled or straight position, re-applying if necessary.

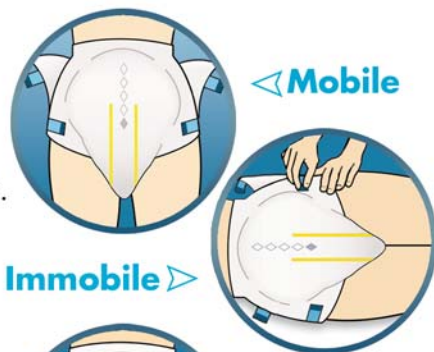

## Step four

- 4** Securely fix the top two tabs in position to create a snug fit at the waist. If necessary, change the positioning to suit the client's body shape.

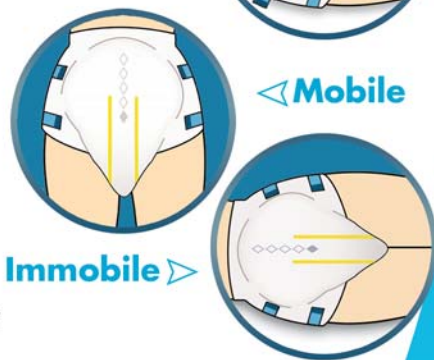

The tabs can be fixed in either an angled or straight position, re-applying if necessary.

## Important Advice

The use of talcum powder is not recommended, oil based barrier creams may reduce the performance of Euron Form pads. If barrier creams must be used they should be applied sparingly and only on treatable areas.

Used products should be disposed of in the appropriate manner. They cannot be flushed down the toilet.



# Euron Form Sizes

|         |              |
|---------|--------------|
| X-Small | 40 - 70 cm   |
| Small   | 50 - 90 cm   |
| Medium  | 80 - 125 cm  |
| Large   | 115 - 155 cm |
| X-Large | 120 - 170 cm |

## Contact Us

ONTEX HEALTHCARE UK LTD  
2nd Floor, Original Wing, Weldon House  
Corbygate Business Park, Priors Haw Road  
Weldon North, Corby, Northamptonshire  
NN17 5JG

TEL: 01536 269744

FAX: 01536 400134

WEB: [www.euronuk.com](http://www.euronuk.com)

**Euron**<sup>®</sup>  
The Same **But Better**

**Ontex**