



Euron® Micro Fitting Guide

Euron®
The Same **But Better**

Euron® Micro

Small Shaped Pads with Adhesive Strip

Euron Micro shaped pads have been designed for the management of (stress) or light incontinence. Featuring a wide adhesive strip, these pads are worn with conventional close-fitting underwear to provide the user with complete discretion

Regardless of the type of Euron Micro product that you will wear the following guidelines apply:

Fitting Instructions

Step one

- 1 Remove the pad from its packaging. Gently unfold the pad, smooth and flatten then halve the pad length ways forming a cupped area in the middle of the pad.

Where the pad is elasticated allow this to aid in shaping and cupping the product. In turn this will help you to achieve a close secure fit.




Step two

- 2 Euron Micro Extra Plus and Super Plus are anatomically shaped to fit closely to your body. When using these pads the wider end should always be placed to the back and the smaller, narrower end to the front.



Step three

- 3 Remove the paper from the adhesive strip running lengthways along the middle of the rear (back) of the pad. Stick the pad onto the crotch area of your pants, carefully positioning it so that any urine that may escape will go into the central area of the pad.

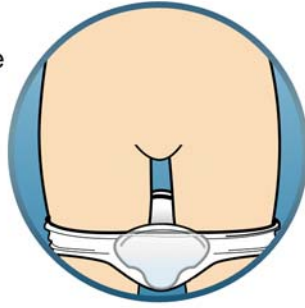

Step four

- 4 Pull your pants up ensuring that your newly applied pad remains in place and fits closely to your groin. Remember always wear your Euron Micro pad with close fitting elasticated underwear.



Important Advice

The use of talcum powder is not recommended, oil based barrier creams may reduce the performance of Euron Micro pads. If barrier creams must be used they should be applied sparingly and only on treatable areas.

Used products should be disposed of in the appropriate manner. They cannot be flushed down the toilet.



Euron Micro Sizes

Product Absorbencies

Ultra



Extra



Super



Extra Plus



Extra Plus



Contact Us

ONTEX HEALTHCARE UK LTD
2nd Floor, Original Wing, Weldon House
Corbygate Business Park, Priors Haw Road
Weldon North, Corby, Northamptonshire
NN17 5JG

TEL: 01536 269744

FAX: 01536 400134

WEB: www.euronuk.com

Euron[®]
The Same **But Better**

Ontex